


Pain Assessment Protocol

1


The Face
Select the Resident you want to do a pain assessment on and Press The "ASSESS PAIN" button

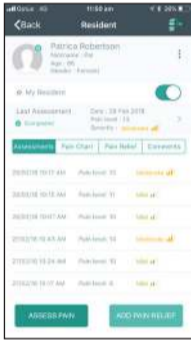


Position the device ~ 1m for the resident

Wait for the face screen to become Active


See Optimal Conditions below



Once screen becomes active press the Start Analysis button within 10 seconds

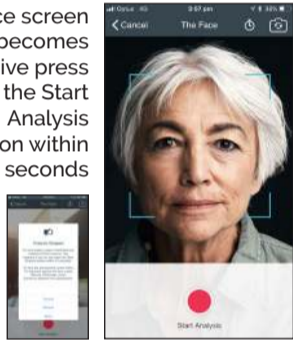


Automated Facial Analysis completed

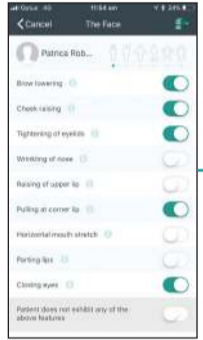


Swipe Left to go to Step 2: The Voice

2

The Voice
Whilst observing the resident "Check" those vocalisations the resident is exhibiting




Swipe Left to go to Step 3: The Movement




3

The Movement
Whilst observing the resident "Check" those movements the resident is exhibiting

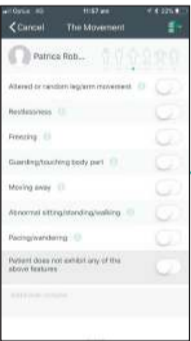


Swipe Left to go to Step 4: The Behaviour




4

The Behaviour
Whilst observing or undertaking clinical review of the resident "Check" those behaviours the resident is exhibiting




Swipe Left to go to Step 5: The Activity




5

The Activity
Whilst observing or undertaking clinical review of the resident "Check" those activities the resident is exhibiting




Swipe Left to go to Step 6: The Body

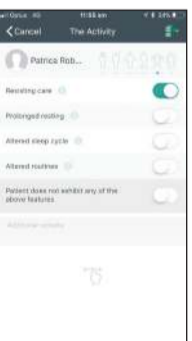


6


The Body
Whilst observing or undertaking clinical review of the resident "Check" those changes in body items the resident is exhibiting

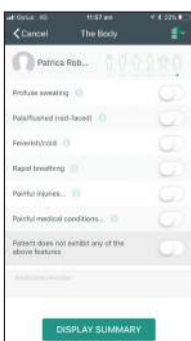


Press DISPLAY SUMMARY

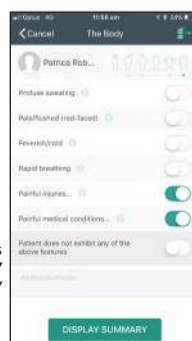


Press SAVE



Press OK

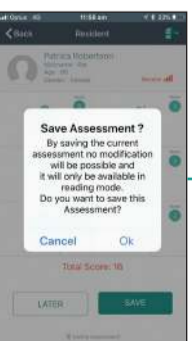

To Monitor Pain Scores



Press Pain Chart

