

Pain Assessment Protocol

1 **The Face**
Select the Resident you want to do a pain assessment on and Press the "ASSESS PAIN" button

Select the activity status and type of facial analysis

Position the device until "the Face" screen becomes active, ensure the device mirrors the angle of the person face

Once screen becomes active press the Start Analysis button within 10 seconds

Automated Facial Analysis completed

When Complete, Swipe Left to go to Step 2: The Voice

2 **The Voice**
Whilst observing the resident "select" those vocalisations the resident is exhibiting

When Complete, Swipe Left to Go to Step 3: The Movement

3 **The Movement**
Whilst observing the resident "select" those movements the resident is exhibiting

When Complete, Swipe Left to Go to Step 4: The Behaviour

4 **The Behaviour**
Whilst observing or undertaking clinical review of the resident "select" those behaviours the resident is exhibiting

When Complete, Swipe Left to Go to Step 5: The Activity

5 **The Activity**
Whilst observing or undertaking clinical review of the resident "select" those activities the resident is exhibiting

When Complete, Swipe Left to Go to Step 6: The Body

6 **The Body**
Whilst observing or undertaking clinical review of the resident "select" those changes in body items the resident is exhibiting

Press DISPLAY SUMMARY

Review the assessment summary to ensure it is complete and accurate, then press SAVE

Press OK

To Review Pain Scores

Information Key

Press Pain Chart